

BinaxNOW[®]

Influenza A & B Rapid Test

See Product Insert for complete use
instructions and performance claims

PART 1 - SAMPLE PREPARATION PROCEDURE

Nasal Wash/Aspirate:

Nasal wash/aspirate samples do not need preparation. Go to PART 2.

Nasopharyngeal and Nasal Swabs:

Elute swab in 0.5-3.0 ml of an appropriate liquid transport system by vigorously rotating the swab in the liquid. Go to PART 2. If eluting swab in the Binax elution solution, follow the Swab Elution procedure below.

Swab Elution (Control and Patient) using Binax Elution Solution:

1. The test kit contains test vials pre-filled with elution solution. Twist off the test vial cap.
2. Put swab to be tested into test vial. Rotate swab vigorously three (3) times in the liquid.
3. Press the swab against the side of the vial and turn as you remove it from the vial. This removes sample from the swab.
4. Discard the swab.
5. Test liquid sample (from the test vial) in the NOW[®] test as soon as possible. Go to PART 2.



Manufactured by:

 **inverness medical**
professional diagnostics

EC REP

EMERGO EUROPE
P.O. BOX 18510
2502 EM THE HAGUE
THE NETHERLANDS



Binax, Inc.
10 Southgate Road, Scarborough, Maine 04074
US: 1-800-323-3199 Outside US: 1-207-730-5700

